

GENERAL NOTES

ALL DIMENSIONS ARE IN MM.  
GENERAL TOLERANCE ON BUILDING ±5MM.


SERVICE REQUIREMENTS (BY OTHERS)

STEAM – SUPPLY TO BE BETWEEN 400 & 500 KPA, AT 0.97 DRYNESS FACTOR, CHLORIDE FREE COMPLETE WITH 20MM ISOLATING VALVE. STEAM CONSUMPTION APPROXIMATELY 50 KG/HR WITH A MAXIMUM FLOW RATE OF 85 KG/HR FOR 3 MINUTES PER CYCLE.

WATER (MAINS) – SUPPLY TO BE CONSTANTLY BETWEEN 220 & 500 KPA, MAXIMUM TEMPERATURE 25°C, CHLORIDE FREE COMPLETE WITH 20MM ISOLATING VALVE & BACKFLOW PREVENTION TO AS 3500.1, AVERAGE WATER CONSUMPTION APPROXIMATELY 13 LITRES/MIN.  
NOTE! WATER CONSUMPTION CAN BE SIGNIFICANTLY REDUCED IF CHILLED WATER IS AVAILABLE.

POWER – 20 AMPS/PHASE, 415 VOLT COMPLETE WITH NEUTRAL, EARTH & ISOLATING SWITCH. FINAL CONNECTION TO UNIT BY ON SITE ELECTRICIAN.

WASTE – 50MM TO SEWER & IN COMPLIANCE WITH AS3500.2. WASTE TEMPERATURE UP TO 100°C SHORT TERM.

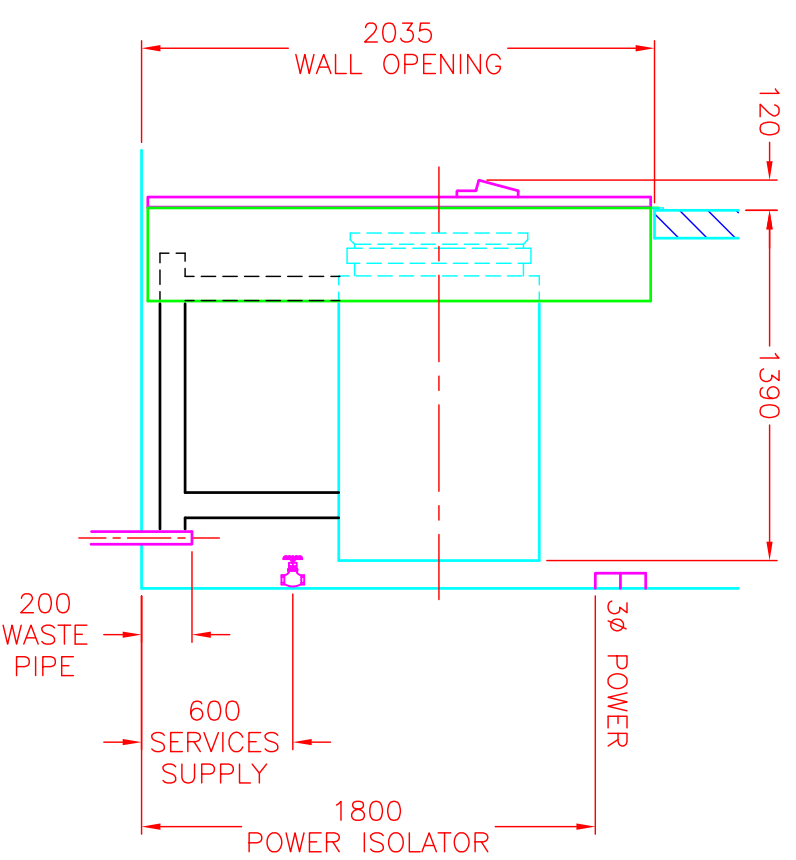

**Atherton**  
HOSPITAL EQUIPMENT  
COMMERCIAL KITCHEN EQUIPMENT  
AIR CONDITIONING & MECHANICAL SERVICES

Head Office, Melbourne Phone: 9497 2500  
Fax: 9497 1190  
Adelaide Phone: 8262 9044  
Sydney Phone: 9679 8911  
Brisbane Phone: 3277 1321  
Perth Phone: 9249 6166  
Web: www.athertononline.net

PROJECT NAME

MAIN TITLE

STERILISER INSTALLATION

SUB TITLE

TSB 6

SUB TITLE

TIGER SERIES

LOCATION

ITEM



PRODUCTION ORDER

DRAWN

H.R

DATE  
13/02/07

SCALE  
1:30

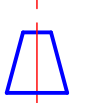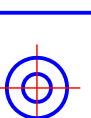

APPROVED BY

STATUS

UNAPPROVED  
 APPROVED BY CLIENT

PENDING APPROVAL  
 APPROVED FOR MANUFACTURE

DRG

SKETCH 1

REV